ILUMARK™ Navigation Marker - Quick Twist
(REF. 2021)

ILUMARK GmbH
Hohenlindner Str. 11 c 85622
Feldkirchen Germany
Tel: +49 (0)89 907 795 98-0
Fax: +49 (0)89 907 795 98-9
Email: info@ilumark.com
www.ilumark.com

CASCINATION AG
Steigerhübelstrasse 3
3008 Bern, Bern
Switzerland

IZI Medical Products, LLC.
5 Easter Court, Suite J
Owings Mills, MD 21117
USA

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This product meets the essential requirements of Medical Device Directive 93/42 EEC.

ENGLISH

**User Instructions for
ILUMARK™ Navigation Marker - Quick-Twist**

Intended use

ILUMARK™ Navigation Markers are intended to be used as image guided surgery system accessory to aid in auto-registration and localization of rigid patient anatomical structure in either open or percutaneous image guided surgery procedures. It is indicated for any medical condition in which the use of an image guided surgery system may be considered to be appropriate.

ILUMARK™ Navigation Markers can be used in following procedures:

- Stereotactic brain surgery
- Endoscopic Sinus Surgery
- Pedicle Screw Placement
- Reconstructive orthopedic surgery
- Soft tissue treatment

⚠ Other applications were not considered and are subject to the responsibility of the user.

ⓘ ILUMARK™ Navigation Markers - Quick-Twist are packed in blue blister packaging. Use these markers only on M3.

Compatibility

ILUMARK™ Navigation Markers - Quick-Twist are compatible with BrainLAB®s navigation systems:

- VectorVision®
- Kolibri™

Instruction for use

Take the ILUMARK™ Navigation Marker out of the inner sterile packaging with sterile, dry and clean gloves and gently place the hole of the sphere over the threaded pin. Screw the sphere onto the pin until there is no visible gap between the bottom side of the sphere and the base of the pin.

⚠ Do not use a ILUMARK™ Navigation Marker, if it is not securely placed in the correct position as shown above.

Only use clean and dry ILUMARK™ Navigation Markers. If the surface of a ILUMARK™ Navigation Marker is soiled or wet, navigation will be inaccurate. To ensure accurate navigation, wet or soiled ILUMARK™ Navigation Markers must either be cleaned or replaced with new ILUMARK™ Navigation Markers. Only use a soft cloth moistened with sterile water to clean the surface of soiled ILUMARK™ Navigation Markers. Thoroughly dry ILUMARK™ Navigation Markers before use.

⚠ If ILUMARK™ Navigation Markers are used on vibrating instruments, regularly verify that they are securely attached.

⚠ If the sterile packaging is damaged or sterility is in doubt, do not use the ILUMARK™ Navigation Markers, but dispose of them.

ⓘ Only physicians experienced with Image Guided Surgery Systems should treat patients with ILUMARK™ Navigation Markers.

Handling

Disposable retro-reflective ILUMARK™ Navigation Markers are highly sensitive medical devices. They should be handled with care and not dropped. Do not use defective or deformed ILUMARK™ Navigation Markers.

Storage

Products must be stored in the original unopened packaging in a dry place. Storage temperature range must be considered.

⚠ Do not re-sterilize ILUMARK™ Navigation Markers!

⚠ Re-sterilization may damage the surface or geometry of the marker sphere. Damages may not be visible to the human eye, but will result in inaccurate navigation and possible patient injuries.

For further information, please contact ILUMARK™ customer support.

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