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**ZH
44**

IZI Medical Products
From concept to care.


T _ D C B N S _ R E V Z

OSTEO-RX™ COAXIAL CURVED BONE BIOPSY AND INFUSION SET

Order Number: DCBNS-13-19.0-C

SET COMPONENTS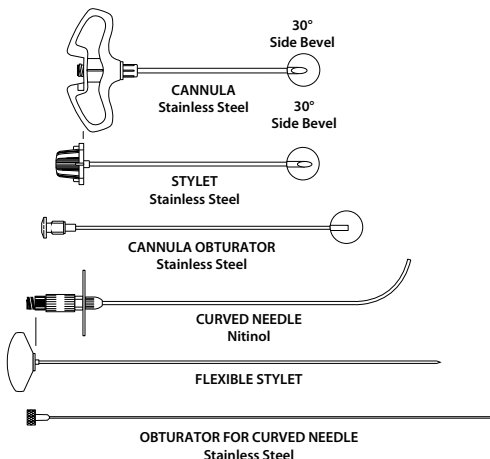

INTENDED USE

The Osteo-Rx™ Coaxial Curved Bone Biopsy and Infusion Set is intended for biopsy and infusion.

This set can be used for:

1. Percutaneous Biopsy - This needle set will allow the user to collect biopsy samples in radial directions around the introducer needle.
2. Infusion - This needle will allow for multiple types of therapeutic materials to be delivered to sites within the bone.

WARNINGS

- Exercise caution when removing curved needle. Spring forces from curved needle may cause injury to the physician or patient.
- Curved needle extraction from the hub may require substantial force. This force will diminish abruptly as the precurved portion is extracted from the hub of the introducer needle. Exercise caution during removal to avoid injury to the physician or patient.

PRECAUTIONS

- Familiarize oneself with set components and interaction of components.
- Assure that inner stylet is in proper alignment with outer cannula for appropriate bevel orientation.
- The curved needle can be difficult to maneuver within the straight introducer needle and may require substantial force to insert or extract.
- Grasp introducer needle cannula firmly during curved needle insertion to prevent further advancement of introducer needle.
- If an obstruction occurs within the curved needle, pull it back so that it assumes a straight position within the introducer needle before introduction of the obturator for the curved needle.
- Be sure to remove the obturator for the curved needle before advancing the curved needle back to desired position.


SUGGESTED INSTRUCTIONS FOR USE OF THE OSTEO-RX™ COAXIAL CURVED BONE BIOPSY AND INFUSION SET

Instructions for Pre-Procedural Familiarization

1. Read and understand all warnings and precautions.
2. Remove needles from package.
3. On a tabletop, remove stylet from introducer needle.
4. Insert the curved needle into the introducer needle to acquaint the user with the coaxial fit of the two needles.
5. Carefully remove the curved needle according to the Instructions for Post-Procedure Removal of Needle. **NOTE:** The curved needle will generate spring forces upon insertion and removal.
6. Replace the initial bevel stylet into the introducer needle.

CAUTION

Assure that inner stylet is in proper alignment with outer cannula for appropriate bevel orientation.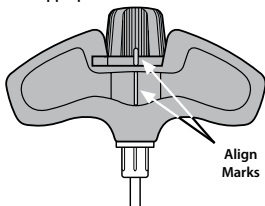

Instructions for Performing Procedure

1. Under fluoroscopic guidance, in both Anterior/Posterior and Lateral views simultaneously, advance the introducer needle to the desired location. **NOTE:** For least amount of resistance, angled bevel face of the introducer needle should be oriented in the direction of the intended curved needle path.

2. Remove stylet.
3. Insert the curved needle into the introducer needle. Do not allow the curved needle tip to exit the introducer needle.

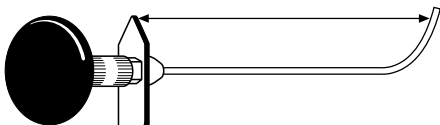
Figure 1

4. Using the arrow shaped hub on the curved needle, align the arrow in the direction desired. (Figure 1)
5. Under fluoroscopic guidance in both Anterior/Posterior and Lateral views simultaneously, advance the curved needle to the desired location. **NOTE:** If needle tip is not oriented in the desired direction, pull the curved needle back into the introducer needle before redirecting and re-advancing.
6. Remove curved needle flexible stylet.
7. Perform biopsy or infusion.

SUGGESTED INSTRUCTIONS FOR POST-PROCEDURE REMOVAL OF NEEDLE

1. Slowly remove the curved needle in a controlled fashion until the curve is fully withdrawn into the introducer needle's cannula.

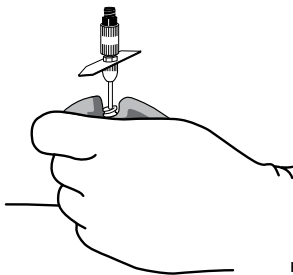
Figure 2

2. Move your hand, which is stabilizing the introducer needle, distally down the handle. (Figure 2)

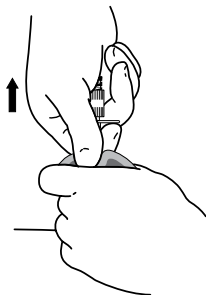
Figure 3

3. Firmly grasp the curved needle's shaft as close to the hub of the introducer needle as possible. (Figure 3)

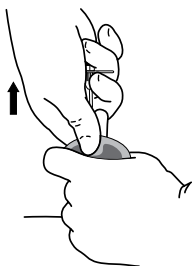
Figure 4a

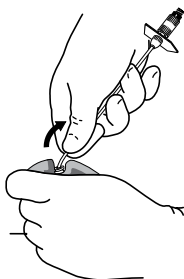
Figure 4b

4. Remove curved needle in small increments, regripping distally when possible until removed. (Figure 4)

WARNING: Curved needle extraction from the hub of the introducer needle may require substantial force. This force will diminish abruptly as the precurved portion is extracted from the hub. Exercise caution during removal to avoid injury to the physician or patient.

PACKAGING/STERILIZATION

Osteo-Rx™ Coaxial Curved Bone Biopsy and Infusion Sets are supplied sterile in peel-open packages. Sterile if package is unopened or undamaged. Store in cool, dry place.

CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

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IZI Medical Products, LLC
ILUMARK GmbH