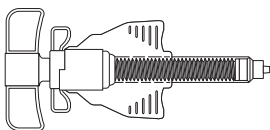
Osteo-Force™ High Pressure Injector Set	EN
Instructions for Use	3
Osteo-Force™ sæt til højtryksinjektion	DA
Brugsanvisning	5
Osteo-Force™ Hochdruckinjektor-Set	DE
Gebrauchsanweisung	7
Σετ εγχυτήρα υψηλής πίεσης Osteo-Force™	EL
Οδηγίες χρήσης	9
Equipo inyector de alta presión Osteo-Force™	ES
Instrucciones de uso	11
Set d'injecteur haute pression Osteo-Force™	FR
Mode d'emploi	13
Set con iniettore ad alta pressione Osteo-Force™	IT
Istruzioni per l'uso	15
Osteo-Force™ hogedrukinjectorset	NL
Gebruiksaanwijzing	17
Conjunto inyector de alta pressão Osteo-Force™	PT
Instruções de utilização	19
Osteo-Force™ högtrycksinjektorset	SV
Bruksanvisning	21
Osteo-Force™ 高压注射器套装	ZH
使用说明	23

 0123

IZI Medical

OSTEO-FORCE™ HIGH PRESSURE INJECTOR SET

Syringe

Connecting Tube

DEVICE DESCRIPTION

The Osteo-Force™ High Pressure Injector Set is a manually powered device used for placing bone cement.

INDICATIONS FOR USE

Intended for placing bone cement into surgical sites.

CONTRAINDICATIONS

None known.

WARNINGS

- To avoid overfilling bone site and to stop the column of cement from advancing, user must turn handle counterclockwise at least one full turn.
- If during injection the user feels an increase in pressure and feels or sees that no cement is being advanced out of the syringe, do not continue injection. Release the pressure and eliminate flow obstruction before continuing.

PRECAUTIONS

- Follow all institutional protocols for the procedure.
- Review the Suggested Instructions For Use booklet prior to using this device.
- Injector is designed to inject viscous materials. Follow the cement manufacturer's instructions for mixing exactly; if the cement is mixed too thin, it may flow back past the plunger.
- Turn the injector handle in a slow and controlled fashion.
- Monitor the injection of cement under fluoroscopic guidance.
- Due to the high pressure that may be generated by this injector, safety glasses and/or a face shield are recommended.
- In order to purge air, hold injector with tip down when advancing plunger toward cement.
- Do not refrigerate device or cement prior to use.

SUGGESTED INSTRUCTIONS FOR USE OF THE OSTEO-FORCE™ HIGH PRESSURE INJECTOR SET

1. Remove the injector/handle and connecting tube from the package.
2. Remove the plunger from the injector barrel.
3. Attach the connecting tube tightly to the Luer lock fitting on the tip of the injector.
4. Mix the bone cement according to the manufacturer's directions.
5. Holding the injector with the tip down, pour the cement in the opening on the back of the injector barrel.
6. While maintaining the vertical position of the injector barrel, insert the plunger.
NOTE: It is important that the injector be held vertically so that all the air will purge past the plunger.
7. When the plunger has been advanced to the level of the cement and the air has been purged, turn the white thread engaging mechanism clockwise 90°, which will engage the plunger threads.
8. Begin advancement of cement by turning the plunger handle in a clockwise direction.
NOTE: Turning the handle one half turn delivers 0.2 ml of cement.
9. Advance cement through the connecting tube in small increments, until connecting tube is free of air. Make sure no cement protrudes past the tip of the connecting tube.
10. Securely attach the connecting tube to the access needle.
11. Slowly turn the plunger in half turn increments, watching under fluoroscopic guidance until an adequate fill has been achieved.
12. Stop the cement flow by turning the handle in a counterclockwise direction at least one full rotation.
NOTE: If it is necessary to reposition needle within bone, relieve pressure before repositioning needle.
13. Disconnect the connecting tube from the needle before removing needle from patient.
NOTE: Do not attempt to refill the injector barrel for another injection. Use a new injector if additional filling is desired.

PACKAGING/STERILIZATION

Supplied in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged.

CAUTION: Federal law (U.S.A.) restricts this device to sale by or on the order of a physician.

STORAGE

Store this product at room temperature in a dry place.

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ILUMARK GmbH